



Molecular Imaging Products Company

A Division of Summit Anesthesia Support
Anesthesia Technologies

ASSEMBLY INSTRUCTIONS FOR STANDARD COMPACT AND COMPACT CONVERTIBLE ANESTHESIA SYSTEM

1. Using 3/16" allen wrench provided, install the bottom upright post into the H-shaped base using the ¼ x 20 x 1 ¼" socket head cap screws provided. Simply slide the lap joint into the base insuring that the colored dots match. Both the H-shaped base and the bottom of upright post have yellow dots in the inside of the lap joints.
2. Using 3/16" allen wrench provided, install the top upright post with shelf into the bottom upright post using the ¼ x 20 x 1 ¼" socket head cap screws provided. Simply slide the lap joints of the posts together insuring that the colored dots match. Both the bottom upright post and the top upright post with shelf have red dots in the inside of the lap joints. NOTE: THIS STEP REQUIRES 2 PEOPLE – ONE TO HOLD THE ASSEMBLY AND THE OTHER TO INSTALL THE SCREWS.

***NOTE: THE SOCKET HEAD CAP SCREWS THAT HOLD THE BASE AND POST ASSEMBLIES TOGETHER MUST BE FIRMLY TIGHTENED.**

*****See picture on back for complete Anesthesia Systems*****



Molecular Imaging Products Company

A Division of Summit Anesthesia Support
Anesthesia Technologies

Standard Compact
AS-01-0001



Compact Convertible
AS-01-0006

